

ACCESSORIES

MBCA



MTC



MCS



MTS



PD ACCESSORIES		
MBCA	BARBED LUER LOCK ADAPTOR	20/BOX
MTC	TITANIUM BARBED LUER LOCK ADAPTOR	5/BOX
MCS	PD CATHETER STYLET	5/BOX
MTS	PD CATHETER TUNNELER	10/BOX
2759BSI	PD STENCIL MAP PACKET	EACH

IMPORTANT RISK INFORMATION

I-SERIES & V-SERIES

Indications for Use: The I-Series / V-Series Peritoneal Dialysis Catheter is indicated for short-term or long-term use where peritoneal access is required for the purpose of peritoneal dialysis for Acute Kidney Injury (AKI) or Chronic Kidney Disease (CKD).

Contraindications: The catheter is not intended for the Tenckhoff trocar method of insertion. The catheter is contraindicated for patients with: Infected anterior abdominal wall. Active peritonitis. Known or suspected allergies to any of the components of the catheter or the kit.

BARBED LUER LOCK ADAPTOR

Indications for Use: The Jet Barbed Luer Lock Adaptor is indicated for long-term use as a connection point for peritoneal dialysis catheters and male luers.

TITANIUM LUER ADAPTOR

Indications for Use: The Jet Titanium Barbed Luer Lock Adaptor is indicated for long-term use as a connection point for peritoneal dialysis catheters and male luers.

PD CATHETER STYLET

Indications for Use: The Jet Peritoneal Catheter Stylet is indicated for transient use to keep a peritoneal dialysis catheter straight and maneuverable during insertion into the peritoneum.

PD CATHETER TUNNELER

Indications for Use: The Jet Peritoneal Catheter Tunneler is indicated for transient use to dissect tissue to create the subcutaneous tunnel opening for the placement of a Medcomp® peritoneal dialysis catheter.

PD ACCESSORY CONTRAINDICATIONS

Known or suspected allergies to the device.

Refer to Instructions for Use provided with the product for complete instructions, warnings, and contraindications. Observe all instructions for use prior to using products. Failure to do so may result in patient complications.

For patient information please visit www.jetmedical.net/patientinformation




Rue Fritz-Courvoisier40
2300 La Chaux-de-Fonds
Switzerland
T: +41-32-9679021 F: +41-32-9679001
www.jetmedical.net

Medical Components, Inc.
1499 Delp Drive, Harleysville, PA 19438, U.S.A.
P: 215.256.4201 F: 215.256.1787
www.medcomp.net

ISO13485
Certified Quality System

PN2753BSI REV. A 5/24

JET PERITONEAL DIALYSIS CATHETERS

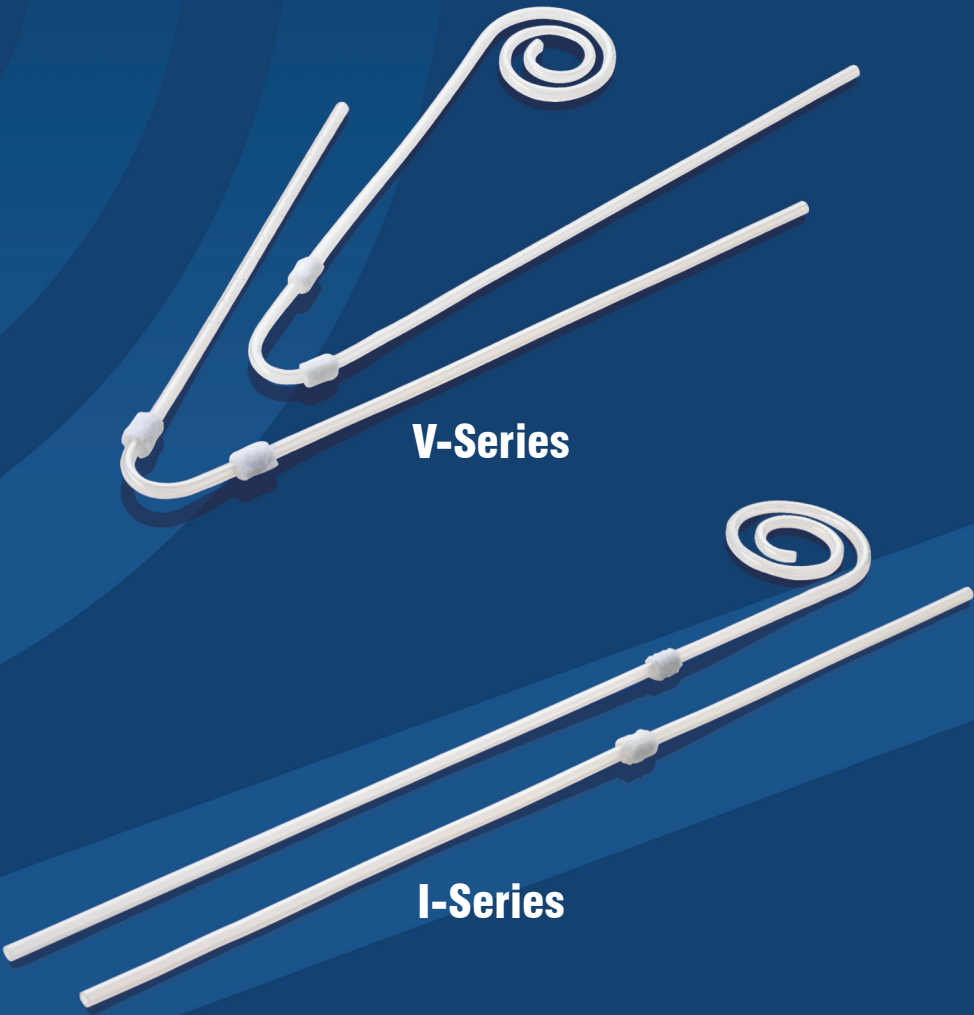

www.jetmedical.net

I-SERIES

Material
Silicone

SIL

Tip Design
Straight & Coiled



French Size
15F



Cuffs
Single & Dual



15F CATHETER ONLY STRAIGHT

Code	Description	Box Qty	Distal Tip to Cuff (CM)	Cuff to Cuff (CM)
JPP41S	15F X 41CM JET I-SERIES CATHETER, SINGLE PREPERITONEAL CUFF	5/BOX	15	-
JSC31S	15F X 31CM JET I-SERIES CATHETER, SINGLE SUBCUTANEOUS CUFF	5/BOX	10.25	-
JSC37S	15F X 37CM JET I-SERIES CATHETER, SINGLE SUBCUTANEOUS CUFF	5/BOX	16.25	-
JSC42S	15F X 42CM JET I-SERIES CATHETER, SINGLE SUBCUTANEOUS CUFF	5/BOX	21	-
JSC46S	15F X 46CM JET I-SERIES CATHETER, SINGLE SUBCUTANEOUS CUFF	5/BOX	25	-
JDC31S	15F X 31CM JET I-SERIES CATHETER, DUAL CUFF	5/BOX	6.5	3
JDC32S	15F X 32.25CM JET I-SERIES CATHETER, DUAL CUFF	5/BOX	7.25	3.5
JDC37S	15F X 37CM JET I-SERIES CATHETER, DUAL CUFF	5/BOX	10.5	5
JDC42S	15F X 42CM JET I-SERIES CATHETER, DUAL CUFF	5/BOX	15	5
JDC47S	15F X 47CM JET I-SERIES CATHETER, DUAL CUFF	5/BOX	20	5

15F CATHETER ONLY COILED

Code	Description	Box Qty	Distal Tip to Cuff (CM)	Cuff to Cuff (CM)	Cuff to Coil (CM)
JPP40C	15F X 40.25CM JET I-SERIES CATHETER, SINGLE PREPERITONEAL CUFF	5/BOX	26	-	16.36
JPP57C	15F X 57.5CM JET I-SERIES CATHETER, SINGLE PREPERITONEAL CUFF	5/BOX	31	-	13
JPP60C	15F X 60CM JET I-SERIES CATHETER, SINGLE PREPERITONEAL CUFF	5/BOX	34	-	15
JDC57C	15F X 57.5CM JET I-SERIES CATHETER, DUAL CUFF	5/BOX	31	6	12
JDC63C	15F X 63CM JET I-SERIES CATHETER, DUAL CUFF	5/BOX	38.25	6	19

15F BASIC SET STRAIGHT

Code	Description	Box Qty	Distal Tip to Cuff (CM)	Cuff to Cuff (CM)
JET-141	15F X 41CM JET I-SERIES CATHETER, SINGLE PREPERITONEAL CUFF	5/BOX	15	-
JET-146	15F X 46CM JET I-SERIES CATHETER, SINGLE PREPERITONEAL CUFF	5/BOX	20	-
JET-146S	15F X 46CM JET I-SERIES CATHETER, SINGLE SUBCUTANEOUS CUFF	5/BOX	25	-
JET-242	15F X 42CM JET I-SERIES CATHETER, DUAL CUFF	5/BOX	15	5

15F BASIC SET COILED

Code	Description	Box Qty	Distal Tip to Cuff (CM)	Cuff to Cuff (CM)	Cuff to Coil (CM)
JET-160	15F X 60CM JET I-SERIES, SINGLE PREPERITONEAL CUFF	5/BOX	34	-	15
JET-257	15F X 57.5CM JET I-SERIES, DUAL CUFF	5/BOX	31	6	12
JET-263	15F X 63CM JET I-SERIES, DUAL CUFF	5/BOX	38.25	6	19

Catheter Only Set Contents: (1) Catheter (1) Female Luer (1) Clamp (1) End Cap

Basic Set Contents: (1) Catheter (1) Introducer Needle (1) Guidewire (1) Guidewire Advancer (1) Syringe (1) Scalpel (1) Tunneler (1) Peelable Introducer (1) Female Luer (1) Clamp (3) Gauze (1) End Cap

V-SERIES

Material
Silicone

SIL

Tip Design
Straight & Coiled



French Size
15F



Cuffs
Dual



15F CATHETER ONLY STRAIGHT

Code	Description	Box Qty	Distal Tip to Cuff (CM)	Cuff to Cuff (CM)
JS200101	15F X 43CM JET V-SERIES CATHETER, LEFT	5/BOX	14.5	5
JS200102	15F X 43CM JET V-SERIES CATHETER, RIGHT	5/BOX	14.5	5

15F CATHETER ONLY COILED

Code	Description	Box Qty	Distal Tip to Cuff (CM)	Cuff to Cuff(CM)	Cuff to Coil(CM)
JS200201	15F X 63CM JET V-SERIES CATHETER, LEFT	5/BOX	34	5	17.82
JS200202	15F X 63CM JET V-SERIES CATHETER, RIGHT	5/BOX	34	5	17.82

15F BASIC SET STRAIGHT


Code	Description	Box Qty	Distal Tip to Cuff (CM)	Cuff to Cuff (CM)
JS200301	15F X 43CM JET V-SERIES CATHETER, LEFT	5/BOX	14.5	5
JS200302	15F X 43CM JET V-SERIES CATHETER, RIGHT	5/BOX	14.5	5

15F BASIC SET COILED

Code	Description	Box Qty	Distal Tip to Cuff (CM)	Cuff to Cuff(CM)	Cuff to Coil(CM)
JS200401	15F X 63CM JET V-SERIES CATHETER, LEFT	5/BOX	34	5	17.82
JS200402	15F X 63CM JET V-SERIES CATHETER, RIGHT	5/BOX	34	5	17.82

Catheter Only Set Contents: (1) Catheter (1) Female Luer (1) Clamp (1) End Cap

Basic Set Contents: (1) Catheter (1) Introducer Needle (1) Guidewire (1) Guidewire Advancer (1) Syringe (1) Tunneler (1) Peelable Introducer (1) Female Luer (1) Clamp (3) Gauze (1) End Cap

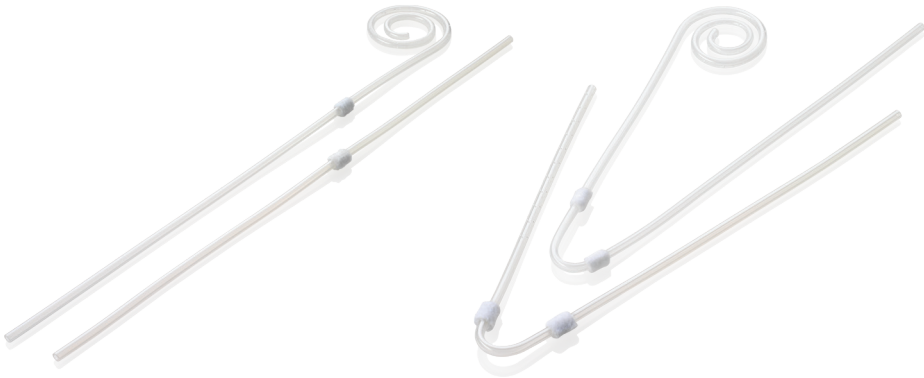

CODES CONTINUED ➔