



Compatibility of Products with Site-Care Agents

This chart indicates the compatibility of our products with commonly used site care formulations. It is neither an all inclusive nor exclusionary listing. Please contact the Customer Service Department with any specific questions concerning site care compatibility. Refer to Instructions for Use provided with the product for complete instructions, warnings, and contraindications. Observe all instructions for use prior to using products. Failure to do so may result in patient complications.

	BRAND NAME	CHLORAPREP®	BETADINE®	N/A	N/A	ANASEPT®	BACTROBAN®	SILVADENE®	PROVIDINE	POLYSPORIN®	N/A
CATHETER NAME	MATERIAL	CHLORHEXIDINE GLUCONATE 2%	POVIDONE IODINE 10%	ISOPROPANOL 70%	HYDROGEN PEROXIDE 3%	0.057% SODIUM HYPOCHLORITE	MUPIROCIN OINTMENT 2%	1% SILVER SULFADIAZINE CREAM	10% POVIDONE - IODINE OINTMENT	BACITRACIN ZINC, POLYMYXIN B SULFATE OINTMENT	GENTAMYCIN SULFATE CREAM 0.1%
JET FLOW	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes
DUO-JET® II	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes